

Prosys®

# Catheter Retaining Strap

**Prosys Catheter Retaining Strap** is designed to keep the catheter or a leg bag tubing in a fixed position and to support the weight of a urinary leg bag.

**Prosys Catheter Retaining Strap** can be worn on the lower abdomen or leg, and its position will depend on whether you have a urethral or suprapubic catheter and also on the length of your catheter and/or leg bag tubing.

## INSTRUCTIONS FOR USE


Place the strap around either the leg or lower abdomen with the silicone beading facing the skin. Fasten the strap in the desired position on your leg or abdomen. Ensure the strap is snug, but not too tight and the Prosys clasp is positioned where it is easily accessible.

- 1 Position the Prosys clasp over the tubing. Wrap the Prosys clasp around and under the tubing.
- 2 Push the tapered end of the Prosys clasp through the eyelet provided.
- 3 Pull the Prosys clasp to gently tighten around the tube. Pull the Prosys clasp up and over to the opposite side, then press the tapered end into place, securing it on the main strap.

Ensure the **Prosys Catheter Retaining Strap** fits tightly enough to keep the strap with the tubing secure during movement, but not too tight that it prevents normal blood circulation.

## Laundering and care:

Close the catheter retaining strap by securing the Prosys clasp in place and wash according to washing instructions. The strap may shrink up to 5% during washing.

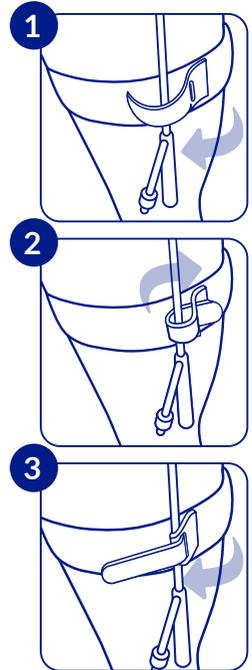

Non-sterile	Keep dry	Batch code	Catalogue number
Consult instructions for use	Keep away from sunlight	Date of manufacture	Medical device
Caution	Do not use if package is damaged	Use by date	Unique device identifier
Not made with natural rubber latex		Manufacturer	UK conformity mark
Authorized representative in the European Community/European Union		European conformity mark	



Clinisupplies Ltd, 1 Blackmoor Lane, Croxley Park, Watford, Hertfordshire WD18 8GA, UK  
Tel: +44 (0)20 8863 4168 Fax: +44 (0)20 8426 0768 Email: info@clinisupplies.co.uk  
www.clinisupplies.co.uk

Advena Ltd, Tower Business Centre, 2nd floor, Tower Street, Swatar, BKR 4013, Malta.