

Attaching a Night Drainage Bag

- 1 Wash your hands thoroughly.
- 2 Ensure the **Prosys® Catheter Valve** Lever is in a closed position and ensure the night drainage bag tap is also closed.
- 3 Remove the cap from the top of the night drainage bag.
- 4 Connect the tubing at the top of the night drainage bag to the drainage tube at the bottom of the **Prosys® Catheter Valve** ensuring the connection is secure.

- 5 Once securely connected open the **Prosys® Catheter Valve** and leave in an open position and allow urine to drain into the night drainage bag.

Please Remember

Empty the bladder at intervals recommended by your doctor or nurse.

	Do not re-use	STERILE EO	Sterilised using ethylene oxide	REF	Catalogue number
	Do not re-sterilise		Caution	LOT	Batch code
	Consult instructions for use		Contains or presence of Phthalates*		Date of manufacture
	Do not use if package is damaged		This device is free from natural latex		Use-by-date

*This product contains the phthalate DEHP. Phthalates are widely used as plasticizers in medical products and have undergone extensive testing. However when using the device, it should be taken into consideration that a lack of evidence regarding DEHP and any disease or adverse effect does not mean that there are no risks.

Prosys® Catheter Valve is a registered trademark of Clinisupplies Ltd



 Clinisupplies Ltd, 1 Blackmoor Lane, Croxley Park, Watford, Hertfordshire WD18 8GA, UK
Tel: +44 (0)20 8863 4168 Fax: +44 (0)20 8426 0768 Email: info@clinisupplies.co.uk
www.clinisupplies.co.uk

EC **REP** Advena Ltd. Tower Business Centre, 2nd floor,
Tower Street, Swatar, BKR 4013. Malta.

CS225 01/21 v4