

Prosys® 2000ml Urine Drainage Bag

Prosys Single Use Night Bag

10 x Drainable, Non-Sterile, 2000ml Night Bag

REF PSU2/PHSU2


Instructions for use

Fitting a Prosys single use night bag to a Prosys leg bag

IMPORTANT: Do not disconnect your leg bag from your catheter

- 1 Open pack.
- 2 Remove the protective cap from the top of the inlet tube on the Prosys night bag. Push the connector into the flexible tube at the bottom of the Prosys leg bag tap.

- 3 Ensure that the tap of the night bag is in CLOSE position as indicated.

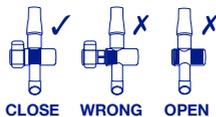


- 4 Once the two bags are securely connected, open the tap of the leg bag, to allow overnight drainage of urine into the night bag.

- 5 To ensure free drainage of urine and support of the Prosys night bag we advise the use of a floor stand or hanger.

Fitting a Prosys single use night bag to a sheath

- 1 Open pack.
- 2 Remove the protective cap from the top of the inlet tube on the Prosys night bag. Push the connector into the sheath outlet ensuring the connection is secure.

- 3 Ensure that the tap of the night bag is in CLOSE position as indicated.

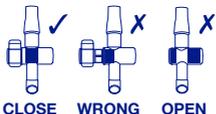


Emptying and disposal of used Prosys single use night bag

- ✓ Remove safety clip on the outlet tap at the bottom of the night bag.

- ✓ Empty the Prosys single use drainage bag by pushing the outlet tap to one side as indicated.

- ✓ The contents should be poured in to the toilet and the toilet flushed.

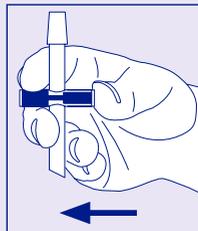

- ✓ If the bags are to be disposed of through the refuse collection service, ensure the bag is placed in a disposable bag and sealed before you place it in a dustbin.

- ✗ Do not incinerate any used bags or any other equipment indoors.

- ✗ Do not flush used bags down the toilet.

IMPORTANT INFORMATION



Non-sterile



Medical device



Do not re-use



Unique device identifier



Consult instructions for use



Batch code



Caution



Manufacturer



Contains or presence of Phthalates



Date of manufacture



Not made with natural rubber latex



Use by date



Keep dry



European conformity mark



Keep away from sunlight



UK conformity mark



Do not use if package is damaged



Authorized representative in the European Community/ European Union



Catalogue number