

# Prosys<sup>®</sup> Urine Drainage Bag












- Take care not to touch the inner contents
- Decontaminate hands before and after procedure by washing and drying thoroughly
- Follow the principles of an aseptic and non-touch technique
- Avoid touching the ends of any connecting areas
- Always adhere to local policies and guidelines

**If the package is damaged, the device must not be used. This device is not suitable for re-sterilisation.**

## Instructions for use

- ① Open Prosys sterile pack taking care not to touch the inner contents
- ② Wash and dry hands thoroughly
- ③ Position and prepare the new Prosys sterile bag. Do not remove the protective cap at this stage
- ④ Disconnect used drainage bag and immediately connect new sterile bag to the catheter or sheath
- ⑤ Avoid touching the ends of any connecting area
- ⑥ Use the new protective cap to cover the end of the used drainage bag
- ⑦ Empty the used drainage bag and dispose of according to local policies and guidelines
- ⑧ Always use an aseptic and non-touch technique
- ⑨ Use on a single patient during single procedure

## SYMBOL EXPLANATION

	<b>Do not re-use</b>		<b>Sterilised using ethylene oxide</b>		<b>Catalogue number</b>
	<b>Do not re-sterilise</b>		<b>Caution</b>		<b>Batch code</b>
	<b>Consult instructions for use</b>		<b>Contains or presence of Phthalates</b>		<b>Date of manufacture</b>
	<b>Do not use if package is damaged</b>		<b>This device is free from natural latex</b>		<b>Use by date</b>