

CLINI



Tubular Bandage

Elasticated Viscose Stockinette

Clini**fast** for Dressing Retention and Wet and Dry Wrapping

Clini**fast** is a light weight, elasticated tubular bandage suitable for dressing retention and for wet and dry wrapping in the treatment of atopic eczema

Dressing Retention

Instructions for Use

- Step 1** Select the correct size and cut to length
 - Step 2** Stretch Clinib**fast** over the affected area
 - Step 3** Place your hand into the bandage and apply to the limb
 - Step 4** Clinib**fast** may be applied in a double layer for extra security over joints
-

Wet and Dry Wrapping:

Instructions for Use

- Step 1** Select Clinib**fast** in the required size to cover the affected area. Measure to ensure full coverage of the area and cut to size
 - Step 2** Before wet or dry wrapping the person should be bathed, dried thoroughly and have had emollient applied to their skin in a downward motion to prevent the hair follicles becoming blocked
 - Step 3** Apply Clinib**fast** to the affected area (knees, wrists, elbows etc)
 - Step 4** When wet wrapping soak in warm water, squeeze out excess and apply damp, follow with a dry top layer
 - Step 5** When dry wrapping apply a single layer only over the emollient which will keep the emollient in place longer
-

Clinib**fast** Sizing Guide

Red Line:	Green Line:	Blue Line:	Yellow Line:	Beige Line:
For small limbs: Unstretched width 3.5cm , limb circumference 8-15cm	For small/medium limbs: Unstretched width 5.0cm , limb circumference 10-25cm	For large limbs: Unstretched width 7.5cm , limb circumference 20-45cm	For children's heads, trunks, extra large limbs and heads: Unstretched width 10.75cm , limb circumference 35-65cm	For adult trunks: Unstretched width 17.5cm , limb circumference 50-120cm

Washing Instructions



HAND
WASH



DO NOT
BLEACH



COOL
IRON



DO NOT
DRY CLEAN



TUMBLE
DRY COOL

Symptoms may improve by using a non-biological washing powder and avoiding the use of fabric conditioner. Seek advice from your healthcare professional.

Storage

Store in dry conditions out of direct sunlight

Fabric Content

92% Viscose, 5% Elastane, 3% Polyamide This device is free from natural latex

Precautions

- Keep Clinifast bandages away from naked flames
- Only wet or dry wrap if advised to do so and instructed by a healthcare professional
- When dry wrapping, always use an emollient
- Do not use if you think the child's skin is infected or the child is unwell and seek immediate medical advice
- Ensure the correct fit to allow complete freedom of movement

For further information on Clinifast bandages for dressing retention and wet or dry wrapping, contact our Customer Services on **020 8863 4168** or visit our website at www.clinisupplies.co.uk.

