

# Prosys<sup>®</sup> Flofit Urinary Sheath

Self Adhesive Silicone Male Urinary Sheath.  
For Single Use. Transparent.

INSTRUCTIONS FOR USE

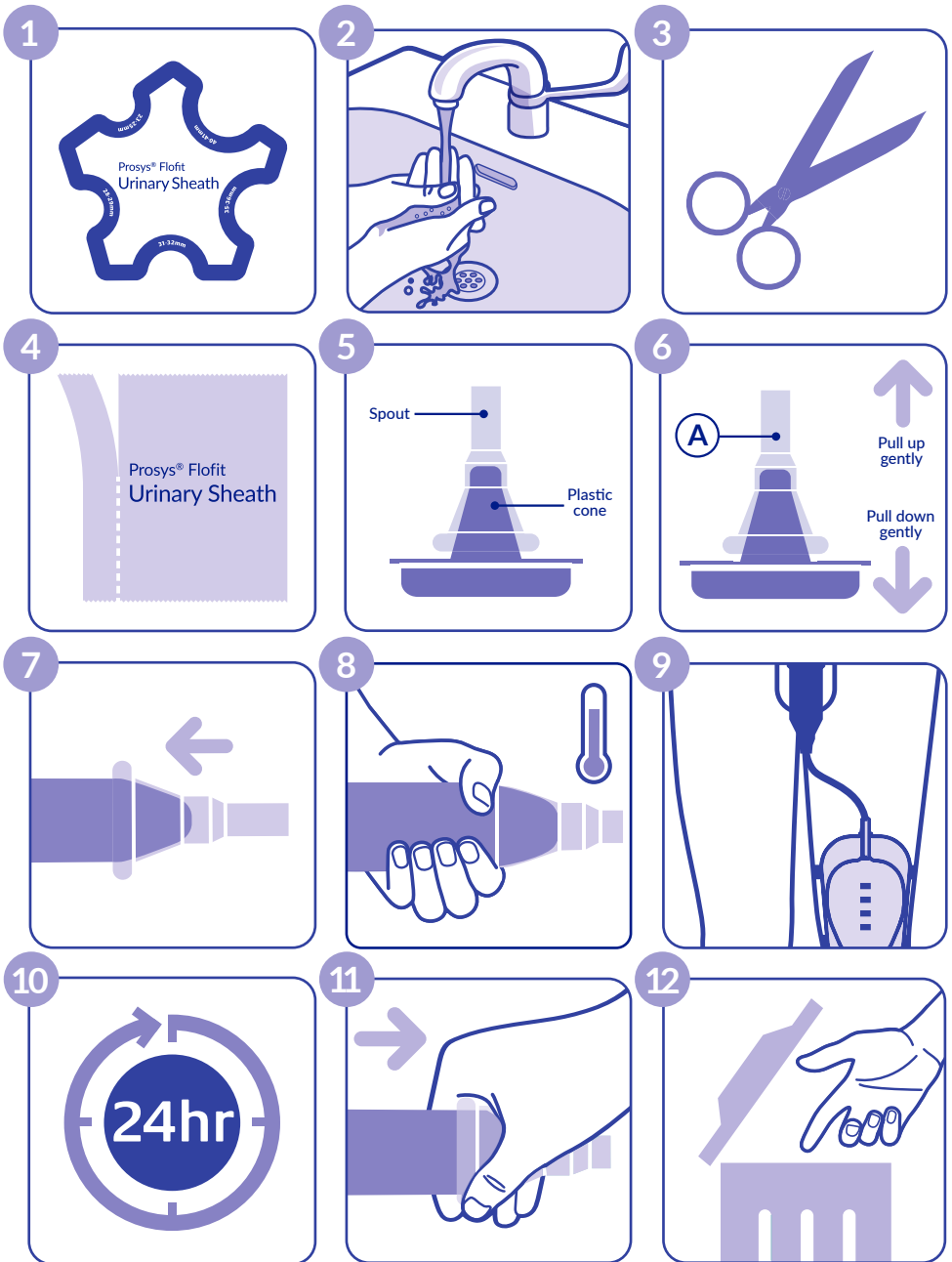
# INSTRUCTIONS FOR USE

## How to fit

- 1 Please ensure you are using the correct size of sheath by utilising the measuring guide enclosed.
- 2 Please ensure you have washed your hands before you use the **Proslys® Flofit Sheath**. Gloves should be worn if the sheath is being fitted by a healthcare professional.
- 3 If necessary, trim any pubic hair or use the hair protectors supplied. Do not shave as this can cause skin irritation. Wash penis and groin area using a mild soap without any moisturiser and ensure the penis and the area around it is completely dry before proceeding. Kitchen roll can be used to dry the skin effectively.
- 4 Open the sheath pouch – tear along the perforation to access the product.
- 5 Remove the sheath from the pack by the spout and clear plastic cone.
- 6 Take the sheath off the plastic cone by holding the sheath just below the spout (**point A on Diagram 6**) and pull down gently on the plastic cone.
- 7 Position the sheath over the penis. If you are using a standard style sheath leave a small space between the end of the penis and the base of the spout – this is the area that allows urine to flow; the adhesive area starts further down the sheath. Unroll the sheath along the shaft of the penis, keeping it even and straight ensuring the end of the sheath is fully unrolled.
- 8 Place your hand around the shaft of the penis and the sheath. Hold the adhesive area for 60 seconds – this will allow the adhesive to warm up and will ensure a secure fit.
- 9 Fit urine drainage leg bag to leg first before connecting to the sheath to avoid any twisting as this may prevent drainage. Ensure the inlet tube is the correct length and not kinked. Only then connect your drainage bag to the spout of the sheath making sure the tap on the bag is in the closed position.

## Removal

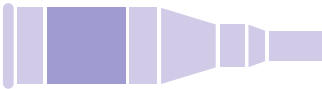
- 10 The sheath can be used for up to a 24hr period to ensure adequate hygiene can be maintained and to allow the skin to breath. Advice should be sought from your healthcare professional if you wish to extend the usage.
- 11 Removal of the sheath may be easier in the bath or shower. Use warm soapy water and wash-cloth and gently roll off the sheath. Alternatively, use an adhesive remover spray to assist removal of the sheath.
- 12 Dispose of your sheath and wash your hands.



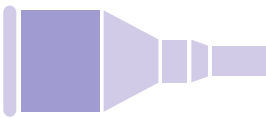
**General advice**

PLEASE NOTE: if your skin becomes irritated whilst using this product, do not use until you have spoken to your healthcare professional. Always check your skin between applications to ensure there is no irritation. Ensure you always dry your skin properly before applying a new sheath.

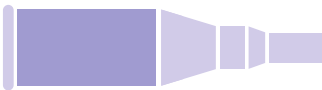
# Self Adhesive Silicone Male Urinary Sheath. For Single Use. Transparent.

STANDARD	STYLE	REFERENCE	SIZE	PACK SIZE
	Style 1	PSS24STD/B	24mm	Box of 30
	Style 1	PSS28STD/B	28mm	Box of 30
	Style 1	PSS31STD/B	31mm	Box of 30
	Style 1	PSS35STD/B	35mm	Box of 30
	Style 1	PSS40STD/B	40mm	Box of 30

A standard length sheath with centrally positioned adhesive. Prosys® Flofit Standard Sheath is suitable for men with an average length of penis.

POP ON	STYLE	REFERENCE	SIZE	PACK SIZE
	Style 2	PSS24POPOPON/B	24mm	Box of 30
	Style 2	PSS28POPOPON/B	28mm	Box of 30
	Style 2	PSS31POPOPON/B	31mm	Box of 30
	Style 2	PSS35POPOPON/B	35mm	Box of 30
	Style 2	PSS40POPOPON/B	40mm	Box of 30

A shorter length sheath with centrally positioned adhesive. Prosys® Flofit Pop-on Sheath is suitable for men with a short or retracted penis.

WIDE BAND	STYLE	REFERENCE	SIZE	PACK SIZE
	Style 3	PSS24WIDEBD/B	24mm	Box of 30
	Style 3	PSS28WIDEBD/B	28mm	Box of 30
	Style 3	PSS31WIDEBD/B	31mm	Box of 30
	Style 3	PSS35WIDEBD/B	35mm	Box of 30
	Style 3	PSS40WIDEBD/B	40mm	Box of 30

A standard length sheath with a greater adhesive area to maximise fit and security. Prosys® Flofit Wide Band Sheath is suitable for men with an average length of penis, who are more active and have trouble keeping other sheaths in place.

## Storage condition

Flofit sheaths must be stored in a cool environment. If the room temperature rises above 18°C, store the sheath in the refrigerator. If the sheath is kept in warm/hot conditions, it can affect the application and result in residual glue on the penis.

	<b>Non-sterile</b>		<b>Do not re-use</b>		<b>Batch code</b>		<b>Catalogue number</b>
	<b>Consult instructions for use</b>		<b>Keep dry</b>		<b>Date of manufacture</b>		<b>Medical device</b>
	<b>Caution</b>		<b>Keep away from sunlight</b>		<b>Use by date</b>		<b>Unique device identifier</b>
	<b>Not made with natural rubber latex</b>		<b>Do not use if package is damaged</b>		<b>Manufacturer</b>		<b>UK conformity mark</b>
	<b>European conformity mark</b>		<b>Authorized representative in the European community/European Union</b>				



 Clinisupplies Ltd, 1 Blackmoor Lane, Croxley Park, Watford, Hertfordshire WD18 8GA, UK  
Tel: +44 (0)20 8863 4168 Fax: +44 (0)20 8426 0768 Email: info@clinisupplies.co.uk  
[www.clinisupplies.co.uk](http://www.clinisupplies.co.uk)

 Advena Ltd, Tower Business Centre, 2nd floor,  
Tower Street, Swatar, BKR 4013, Malta.