

CLINI



Tubular Bandage

Elasticated Viscose Stockinette

Clinifast for Dressing Retention and Wet and Dry Wrapping

Clinifast is a light weight, elasticated tubular bandage suitable for dressing retention and for wet and dry wrapping in the treatment of atopic eczema

Dressing Retention

Instructions for Use

- Step 1** Select the correct size and cut to length
 - Step 2** Stretch Clinifast over the affected area
 - Step 3** Place your hand into the bandage and apply to the limb
 - Step 4** Clinifast may be applied in a double layer for extra security over joints
-

Wet and Dry Wrapping:

Instructions for Use

- Step 1** Select Clinifast in the required size to cover the affected area. Measure to ensure full coverage of the area and cut to size
 - Step 2** Before wet or dry wrapping the person should be bathed, dried thoroughly and have had emollient applied to their skin in a downward motion to prevent the hair follicles becoming blocked
 - Step 3** Apply Clinifast to the affected area (knees, wrists, elbows etc)
 - Step 4** When wet wrapping soak in warm water, squeeze out excess and apply damp, follow with a dry top layer
 - Step 5** When dry wrapping apply a single layer only over the emollient which will keep the emollient in place longer
-

Clinifast Sizing Guide

| Red Line: | Green Line: | Blue Line: | Yellow Line: | Beige Line: |
|--|--|---|---|--|
| For small limbs: Unstretched width 3.5cm , limb circumference 8-15cm | For small/medium limbs: Unstretched width 5.0cm , limb circumference 10-25cm | For large limbs: Unstretched width 7.5cm , limb circumference 20-45cm | For children's heads, trunks, extra large limbs and heads: Unstretched width 10.75cm , limb circumference 35-65cm | For adult trunks: Unstretched width 17.5cm , limb circumference 50-120cm |

Washing Instructions



HAND
WASH



DO NOT
BLEACH



COOL
IRON



DO NOT
DRY CLEAN



TUMBLE
DRY COOL

Symptoms may improve by using a non-biological washing powder and avoiding the use of fabric conditioner. Seek advice from your healthcare professional.

Storage

Store in dry conditions out of direct sunlight

Fabric Content

92% Viscose, 5% Elastane, 3% Polyamide This device is free from natural latex

Precautions

- Keep Clinifast bandages away from naked flames
- Only wet or dry wrap if advised to do so and instructed by a healthcare professional
- When dry wrapping, always use an emollient
- Do not use if you think the child's skin is infected or the child is unwell and seek immediate medical advice
- Ensure the correct fit to allow complete freedom of movement

For further information on Clinifast bandages for dressing retention and wet or dry wrapping, contact our Customer Services on **020 8863 4168** or visit our website at www.clinisupplies.co.uk.

| | | | | | |
|--|---|--|--------------------------------|--|---------------------------------|
| | Non-sterile | | Keep dry | | Catalogue number |
| | Consult instructions for use | | Keep away from sunlight | | Medical device |
| | Caution | | Batch code | | Unique device identifier |
| | Not made with natural rubber latex | | Date of manufacture | | Manufacturer |
| | Do not use if package is damaged | | Use by date | | UK conformity mark |
| | Authorized representative in the European Community/European Union | | | | European conformity mark |



Clinisupplies Ltd,
1 Blackmoor Lane, Croxley Park, Watford, Hertfordshire WD18 8GA, UK
Telephone: 020 8863 4168 Fax: 020 8426 0768
Email: info@clinisupplies.co.uk www.clinisupplies.co.uk

Clinisupplies



Advena Ltd, Tower Business Centre, 2nd floor,
Tower Street, Swatar, BKR 4013, Malta.